


IMPORTANT


You Must Always Hold Handle at End Nearest Cord

Holding handle near electrode end may cause a discharge of current through handle, thus detracting from current delivered for treatment through electrode.


TO INSERT OR REMOVE ELECTRODE FROM HANDLE

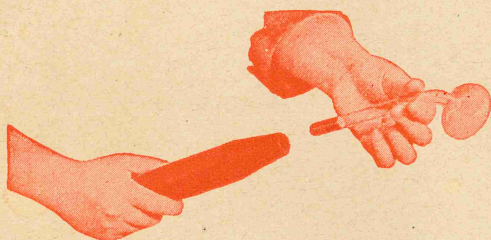

1st—Grasp electrode with thumb and first finger of left hand and applicator handle in right hand as shown in upper photo.

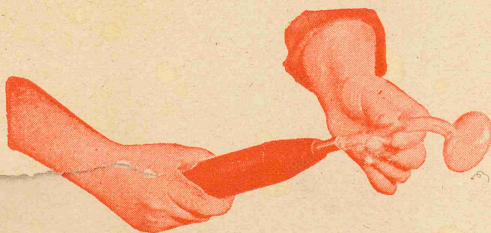

2nd—Insert electrode into handle until electrode strikes electrode holder. (Slight resistance is felt).

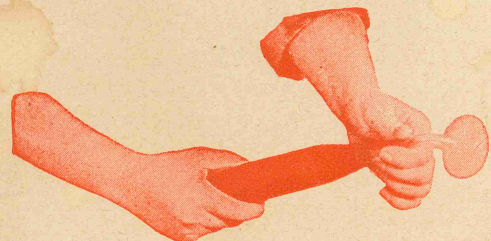

3rd—Press electrode firmly into place.

TO REMOVE REVERSE THE ABOVE OPERATION

DON'T TWIST ELECTRODES
while inserting or removing
from handle

Renulife Electric Company

Detroit,

-

Michigan